



# INSTRUCTIONS FOR USE WITH ADAPTER CAP AND ORAL SYRINGE

## CONTENTS

### ADAPTER CAP\*

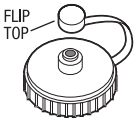


### ORAL SYRINGE

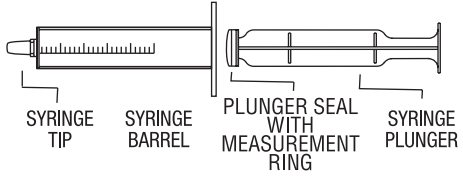

**\*WARNING: ADAPTER CAP IS NOT CHILD-RESISTANT.**


**Keep out of reach of children.**

Please note product images are for illustrative purposes only. Actual product may vary.

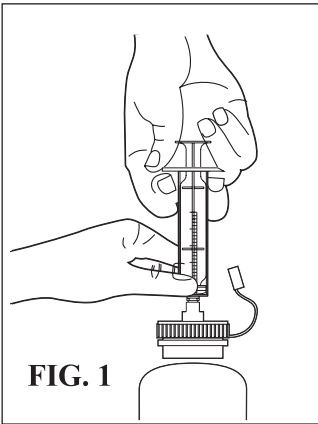

**FIG. 1**

1. Remove the existing white cap on the bottle.
2. Twist the blue adapter cap securely into place on the top of the bottle.
3. Ensure the syringe plunger is fully inserted into the syringe tip. Insert syringe tip into the blue adapter cap (**FIG. 1**).

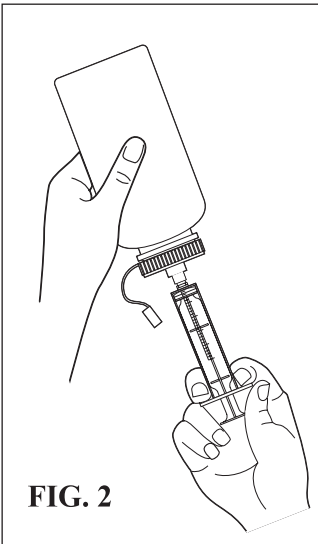

**FIG. 2**

4. Carefully turn the bottle upside down and pull the plunger back to draw the prescribed dose into the syringe (**FIG. 2**).
5. Turn the bottle back upright and carefully remove the oral syringe from the adapter cap, holding it by the barrel rather than the plunger.
6. After administering the dose, snap the flip top lid onto the hole of the adapter cap.
7. Rinse the oral syringe in warm water and leave to dry.

For additional information or questions, please contact us; 800-461-7449, or email us at [customerservice@azurity.com](mailto:customerservice@azurity.com)

Manufactured for:  
**azurity**<sup>®</sup>  
pharmaceuticals

Wilmington, MA 01887 USA

[www.azurity.com](http://www.azurity.com)

These instructions may have been updated. For current instructions for use with adapter cap and oral syringe, please visit [www.FIRSTKITS.com](http://www.FIRSTKITS.com)

PN: 65628-00536 Rev.00